EN 3 Amplatz Needle Holder with Silicone Inserts

Instructions for Use

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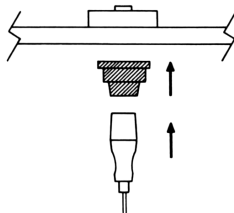
Figure C

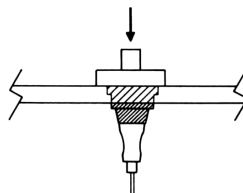
Figure D

AMPLATZ NEEDLE HOLDER WITH SILICONE INSERTS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Polycarbonate needle holder
- Silicone inserts

NOTE: Set components may vary.

INTENDED USE

This device is used to direct a Cook standard disposable needle during needle access procedures. It prevents the stylet from backing out of the cannula during puncture and also keeps the hands from radiation exposure during the puncture procedure.

CONTRAINDICATIONS

None known

WARNINGS

Needles used with this device should always be used under fluoroscopic guidance.

PRECAUTIONS

Always ensure the needle is securely placed within the silicone insert.

INSTRUCTIONS FOR USE

1. Secure the Cook trocar needle in the silicone insert within the clear plastic needle handle.
2. Position your hands to the outside of the device to ensure minimal radiation exposure during the procedure.

Cleaning, Disinfection and Sterilization

NOTE: All cleaning, disinfection and sterilization methods listed herein have been validated.

To Clean

1. Disassemble the Amplatz Needle Holder by removing the silicone inserts from the device.
2. Remove all residual organic matter, blood and irrigation solution to allow the sterilization or disinfection medium to contact the surface of the device. **NOTE:** Ethylene oxide gas and disinfection solutions do not effectively penetrate residual organic matter.
3. Prepare a mild, neutral-pH enzymatic detergent (i.e., Enzol using 1 ounce per gallon) and warm (30-43°C) tap or Deionized (DI) water. Immerse disassembled device and soak for 20 minutes. Use a soft brush to remove all residual organic matter, blood and irrigation solution from the device.
4. Prepare a fresh, neutral-pH enzymatic detergent (i.e., Enzol using 1 ounce per gallon) and warm (30-43°C) tap or DI water. Immerse disassembled device and perform ultrasonic cleaning for 20 minutes.
5. Rinse thoroughly with clean running tap or DI water.
6. Dry all exterior surfaces with a soft cloth.
7. Prior to disinfection/sterilization and reuse, this product should be inspected and reassembled to verify that it functions properly. All surfaces should be smooth and free of abrasions. The device must be disassembled again prior to disinfection and/or sterilization.

NOTE: Automated cleaning methods are not compatible with the Amplatz Needle Holder.

To Disinfect

NOTE: CIDEX® Solution is a high-level disinfection solution that has been found to be compatible with the Amplatz Needle Holder.

1. Disassemble the Amplatz Needle Holder by removing the silicone inserts from the device.
2. Activate CIDEX® Solution per the manufacturer's instructions. Test the activated solution prior to use with CIDEX® Solution Test Strips.
3. Immerse the disassembled device completely, filling all hard-to-reach areas and eliminating air pockets, in CIDEX® Activated Dialdehyde Solution for a minimum of 45 minutes at 25°C.
4. Rinse thoroughly in a room temperature Sterile Water for Injection (SWFI) water bath. Rinse by immersing the device completely for a minimum of one minute. Manually flush all hard-to-reach areas with SWFI water.
5. Agitate the device under SWFI water, bring above water level, then re-immerses.
6. Repeat steps 4 and 5 two additional times using fresh SWFI for each of the three rinses.

NOTE: Automated disinfection methods are not compatible with the Amplatz Needle Holder.

To Sterilize

NOTE: Autoclave sterilization is not recommended for the Amplatz Needle Holder.

This device can be sterilized by ethylene oxide (EO) gas. EO sterilization should be performed in accordance with industry standards with a validated sterilization cycle. The device must be disassembled prior to sterilization. The recommended cycle for sterilizing this device using 100% EO in a double-pouched configuration is:

Temperature = 54.4°C

Humidity = 50-80%RH

EO concentration = 735 mg/L EO

Gas dwell time = minimum 60 minutes

SIX-MONTH LIMITED WARRANTY

Cook Incorporated (Company) warrant to the purchaser of this product that at the time of manufacture, the product was prepared and tested in accordance with Good Manufacturing Practices specified by the United States Food and Drug Administration and was true to label.

In the unlikely event of product failure due to defects in material or workmanship within a period of six (6) months from the date of purchase, the product will be repaired, or at our option, replaced, at no charge. The product must be returned clean, sterile, and properly packaged for shipment. This limited warranty does not apply to products damaged by accident, misuse, improper line voltage, or to products altered or serviced by anyone other than Cook Urological, Incorporated or Cook Ob/Gyn®.

Before returning a product for any reason, please call toll-free for assistance and instructions.

Because we have no control over the conditions under which this product is used, diagnosis of the patient, anatomical or biological differences among patients, method of use or administration, and handling of the product after it leaves our possession, we do not warrant a good effect or against an ill effect following the product's use. The foregoing warranty is exclusive and in lieu of all other warranties whether written, oral, or implied. **There are no warranties of merchantability or fitness for a particular purpose.** No representative of the Company may change any of the foregoing, and the buyer accepts the product subject to all terms hereof. We reserve the right to change or discontinue this product without notice.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Reusable. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.