COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

COOK CollectEvac™

EN

3

CS

4

DA

5

NL

6

FI

7

FR

8

DE

10

EL

11

HU

12

IT

13

JA

14

NO

15

PL

16

PT

17

ZH

18

ES

19

SV

20

COOK® CollectEVAC™

For use with ACMI, STORZ and Wolf resectoscopes

REFERENCE GUIDE

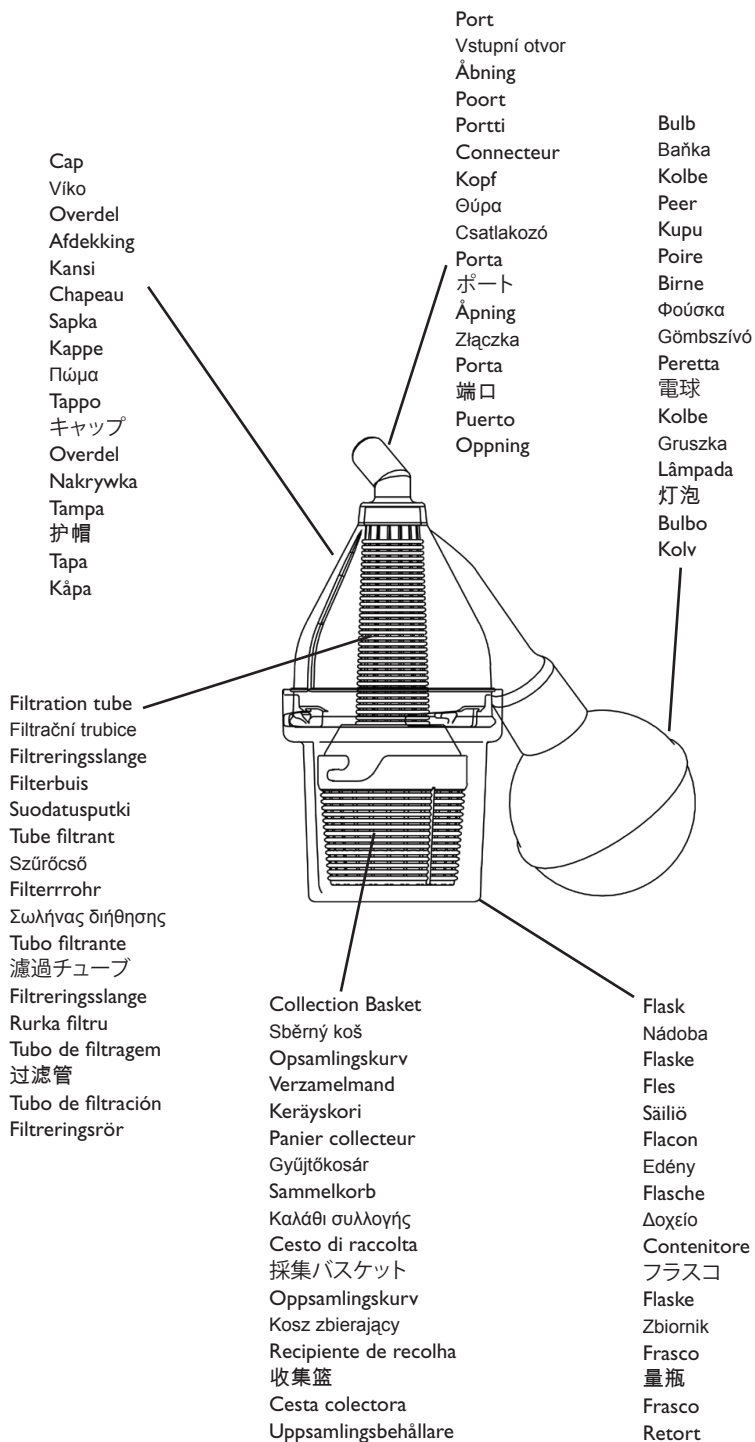
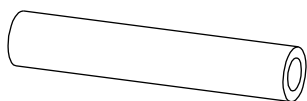
Distributed By

Cook Incorporated
750 Daniels Way
Bloomington, Indiana
47404 USA


CE
0120

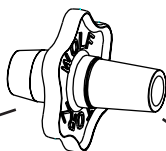Surgical Tubing
 Chirurgicalká hadice
 Operationsslange
 Chirurgische buizen
 Kirurginen putkitus
 Tubulure chirurgicale
 Chirurgisches Rohr
 Χειρουργική σωλήνωση
 Sebészi szerelék
 Tubo chirurgico
 手術用チューブ
 Operasjonsrør
 Kaniula
 Tubagem cirúrgica
 手术管路
 Tubo quirúrgico
 Operationsrør

ACMI adapter
 Adaptér ACMI
 ACMI tilkoblingsled
 ACMI-adapter
 ACMI-adapteri
 Adaptateur ACMI
 ACMI Adapter
 Προσαρμογέας ACMI
 ACMI csatlakozó
 Adattatore ACMI
 ACMIアダプタ
 ACMI tilkoblingsledd
 Złącze ACMI
 Adaptador ACMI
 ACMI 适配器
 Adaptador ACMI
 ACMI-adapter

STORZ/Wolf adapter
 Adaptér STORZ/Wolf
 STORZ/Wolf tilkoblingsled
 STORZ/Wolf-adapter
 STORZ/Wolf-adapteri
 Adaptateur STORZ/Wolf
 STORZ/Wolf Adapter
 Προσαρμογέας STORZ/Wolf
 STORZ/Wolf csatlakozó
 Adattatore STORZ/Wolf
 STORZ/Wolf アダプタ
 STORZ/Wolf tilkoblingsledd
 Złącze STORZ/Wolf
 Adaptador STORZ/Wolf
 STORZ/Wolf 适配器
 Adaptador STORZ/Wolf
 STORZ/Wolf-adapter

**INTENDED
USE**

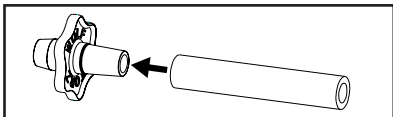
- This device is intended to be used to perform irrigation, tissue collection and/or straining functions during transurethral prostate or bladder surgery.

WARNING


- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Do not use if device appears damaged.*

Assembling COOK CollectEVAC

1. Select corresponding adapter end for use with either ACMI or STORZ/Wolf resectoscopes. Attach opposite end of adapter to surgical tubing.

2. Attach surgical tubing to port. Ensure tubing is secure.

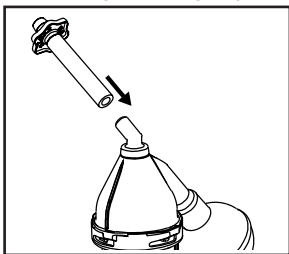**Using COOK CollectEVAC**

3. Prior to use, ensure the following:
 - Collection basket connects to the filtration tube.
 - Bulb securely attaches to cap.
 - Cap securely attaches to flask by turning clockwise.
4. Fill device with fluid.
5. Prior to use, purge all air from device which can be achieved in several ways.
6. To disassemble, remove flask from cap by turning counter-clockwise while holding device over basin. Remove collection basket.

NOTE

- Collection basket may be placed in pathology processing jar once all tissue has been retrieved.
- Refer to steps 3-5 if reassembly is required to complete procedure.

Disposal**WARNING**

- *Dispose of all components as infectious waste.*
- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

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COPYRIGHT © 2008 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. COLLECTEVAC IS A TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. PATENT PENDING.
MPS GmbH