Cook® Cystostomy Catheter

Cystostomický katetr Cook®

Cook® cystostomikateter

Cook® Zystostomie-Katheter

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T _ C C O C _ R E V O

COOK® CYSTOSTOMY CATHETER

Used for temporary suprapubic urinary diversion and drainage. Intended for one-time use.

DESCRIPTION

- Silicone catheter
- Stylet with access sheath
- Drainage adapter with stopcock
- Soft low-profile catheter support with pull tie

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING COOK® CYSTOSTOMY CATHETER

1. Fill the bladder with sterile saline solution if not already distended with urine.
2. Use stylet/access sheath to puncture bladder.
3. If bladder pressure is low and urine fails to flow spontaneously through the stylet handle, aspirate urine to ascertain whether the stylet/access sheath is within the bladder.
4. Leaving access sheath in place, remove stylet.
5. Insert silicone catheter (distal tip has sideports) through access sheath to desired location.
6. Remove and discard access sheath.
7. Attach drainage adapter to catheter.
8. Using the pull tie, attach the catheter support to the catheter shaft 1 to 2 cm from the skin surface. Use suture or surgical tape to fix the catheter support in position and prevent the catheter from being accidentally displaced forward.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.