EN
3

Cook® Drainage Pouch

Instructions for Use

CS
5

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DA
7

Cook® drænagepose

Brugsanvisning

DE
9

Cook® Drainagebeutel

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EL
11

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COOK® DRAINAGE POUCH

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The Cook Drainage Pouch was created to meet the needs of ambulant patients who prefer not to use a leg bag. The pouch is available in set form for unilateral or bilateral application. Each set includes a belt with adjuster, appropriate inlet tubing with a Luer lock connector at the proximal end, and pouch. The pouch utilizes a reed valve anti-reflux mechanism to prohibit reflux of urine. All component parts are available separately.

INTENDED USE

The Cook Drainage Pouch is designed for collection of nephrostomy and biliary drainage.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Care must be exercised when determining appropriate connecting tube length; improper length may cause the tube to kink, resulting in urine reflux to the kidney(s). Periodic examination of the connecting tube is recommended.
- This pouch is intended for single patient use only.
- Do not allow the pouch to exceed maximum capacity.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

1. The drainage pouch should be worn over the central lower abdomen and held in place with moderate belt tension. **Always position the pouch below the nephrostomy/biliary site for proper drainage.**

NOTE: The pouch may be worn over underwear to reduce perspiration.

2. The connecting tube may be cut to fit the patient. Disconnect the tube at the pouch, remove excess tubing from the distal end, and replace. **CAUTION: Care must be exercised when determining appropriate connecting tube length; improper length may cause the tube to kink, resulting in urine reflux to the kidney(s). Periodic examination of the connecting tube is recommended.**

Cleaning the Pouch

Repeated use of the pouch is possible; however, daily cleaning of the pouch and belt is recommended. Cook recommends the following cleaning procedure and wearing times if approved by the patient's physician. Consult your physician before attempting to remove and clean this device.

NOTE: Two nephrostomy sets, used interchangeably, allow a fresh set to be used each day.

1. Remove the empty nephrostomy set and rinse thoroughly with lukewarm water.
2. Soak the nephrostomy set for 30 minutes in a solution of one part white vinegar to two parts water. A syringe (without the needle attached) may be used to clean and rinse the connecting tube.
3. The bag and connector must be completely submerged while soaking to ensure thorough cleaning inside and out.
4. Rinse thoroughly in lukewarm water and hang to dry.

- Wash the belt strap by hand in lukewarm water using a mild soap. Rinse thoroughly and hang to dry.

Wearing Times

With proper care, the appliance may be worn 10-14 days. Wearing times may vary depending on body metabolism. Cook recommends the following wearing times if approved by the patient's physician. It is recommended that two appliances be used interchangeably so a fresh appliance can be applied daily.

Non-Sterile Order Number	Description
505003	300 ml Pouch Set with Single Connector
505004	300 ml Pouch Set with Double Connector
505011	500 ml Pouch Set with Single Connector
505012	500 ml Pouch Set with Double Connector
505005	Single Connector with Connecting Tube
505006	Double Connector with Two Connecting Tubes
053003	Foley Adapter

Sterile Order Number	Description
515000	300 ml Pouch
515003	300 ml Pouch Set with Single Connector
515004	300 ml Pouch Set with Double Connector
515011	500 ml Pouch Set with Single Connector
515012	500 ml Pouch Set with Double Connector
515001	Belt and Adjuster
515005	Single Connector with Connecting Tube
515006	Double Connector with Two Connecting Tubes
053003	Foley Adapter

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.