EN
3

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ZH
20

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T _ C L S T _ R E V O

CLEANING AND STERILIZATION INSTRUCTIONS

1. Remove all residual organic matter, blood and irrigation solution to allow the sterilization or disinfection medium to contact the surface of the device.

NOTE: Ethylene oxide (ETO) gas and disinfection solutions do not effectively penetrate residual organic matter.

2. Clean all surfaces with a solution of mild detergent and water. Use a soft brush to remove all residual organic matter, blood and irrigation solution from all surfaces.
3. Ultrasonic cleaning of the components may be performed. Ultrasonic cleaning should only be performed with non-abrasive and non-corrosive solutions.
4. Rinse thoroughly with clean water.
5. Dry all exterior surfaces with a soft cloth or an air pistol.
6. Dry all openings with an air pistol.
7. Prior to sterilization and reuse, inspect device to verify that it functions properly. All edges and surfaces should be smooth and free of abrasions.

ETHYLENE OXIDE GAS STERILIZATION

Routine ethylene oxide (ETO) gas sterilization is the sterilization method of choice.

ETO gas sterilization and aeration should be performed in accordance with industry standard with a validated sterilization cycle.

AUTOClave STERILIZATION

This device can be sterilized by steam autoclave. Autoclave sterilization should be performed in accordance with industry standard. Autoclave temperature should not exceed 275°F.

DISINFECTION

- Chemical sterilization should be performed in accordance with industry standard.
- Only hard surface disinfectants capable of high-level disinfection should be used.
- Carefully follow the disinfectant manufacturer instructions.
- Devices to be disinfected should be placed in a plastic basin to avoid scratches and electrolytic corrosion.
- Rinse disinfected devices thoroughly to remove all traces of residue.

LIMITED WARRANTY STATEMENT

Cook Urological Incorporated ("Company") warrants to the purchaser of this product (the "Product") that at the time of manufacture, the Product was manufactured and tested in accordance with Good Manufacturing Practices specified by the United States Food and Drug Administration and was true to label. In the unlikely event of a Product failure due to defects in material or workmanship within the warranty period, the Product will be repaired or replaced, at Cook's option, free of charge.

Because the Company has no control over the conditions under which this Product is used, diagnosis of the patient, anatomical or biological differences among patients, method of use or administration, or handling of the Product after it leaves the Company's possession, the Company does not warrant a good effect or against an ill effect following the Product's use. The foregoing warranty is exclusive and in lieu of all other warranties whether written, oral, or implied. There are no warranties of merchantability or fitness for a particular purpose. No representative of the Company may change any of the foregoing, and the purchaser accepts the Product subject to all terms hereof. The Company reserves the right to change or discontinue this Product without notice.

Limits and Exclusions

The warranty does not apply in the event of misuse, negligence, or accidental damage of Product.

Warranty services will be provided only to the original purchaser at the location where the Product was originally installed. The warranty may not be transferred without the prior written consent of the Company.

Warranty Period

This limited warranty begins on the date the Product ships from the Company's facilities and ends 6 months later.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.