Flexor® Ureteral Access Sheath

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T _ F L X R _ R E V O

FLEXOR® URETERAL ACCESS SHEATH

Used to establish a conduit during endoscopic urological procedures facilitating the passage of endoscopes and other instruments into the urinary tract. AQ® hydrophilic coating is applied to the dilator/sheath assembly to ease placement. Intended for one-time use.

DESCRIPTION

- Ureteral access sheath
- Tapered dilator with locking hub

NOTE: Thirteen (13) cm length is intended for use in pediatric patients two (2) years of age and over.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING FLEXOR® URETERAL ACCESS SHEATH

1. Place an .038 inch (0.97 mm) diameter wire guide the desired length into the ureter to establish a working tract.
2. Grasp the sheath just below the instrument adapter and advance the dilator/sheath assembly over the wire guide and into the ureter. **NOTE: Be sure the dilator is securely locked onto the instrument adapter, ensuring the dilator/sheath assembly can be placed as a single unit, allowing one-hand placement.**
3. Confirm the dilator/sheath assembly is properly placed via fluoroscopy.
4. While holding the Flexor® sheath in position, unlock the fitting and remove the dilator.
5. Introduce the desired endoscope or instrument as needed.

NOTE: Suture may be utilized to secure the adapter externally. Suture holes are conveniently located on the instrument adapter.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.