Balloon Uterine Stent

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Ballonstent til uterus

Ballon-Uterusstent

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BALLOON UTERINE STENT

Used for placement in the uterus following an intrauterine surgical procedure in order to reduce uterine bleeding. Intended for one-time use.

DESCRIPTION

- Silicone stent with balloon
- Syringe

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

Use of this device is contraindicated in the presence of pregnancy, cervical cancer, purulent infections in the vagina, cervix or uterus, untreated uterine anomaly, or a surgical site which would prohibit the device from effectively controlling bleeding.

SUGGESTED INSTRUCTIONS FOR USING BALLOON UTERINE STENT

1. Roll the balloon longitudinally to the smallest diameter possible and grasp the top of the rolled balloon with a smooth forceps.
2. Insert the forceps tip with the balloon through the cervical canal to the uterine cavity above the internal os of the cervix.

WARNING: Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

3. Partially inflate the balloon.
4. Gently withdraw the smooth forceps leaving balloon in place.
5. Inflate the balloon to maximum inflation volume as stated on product label.

WARNING: Do not overinflate the balloon. Refer to product label for maximum inflation volume.

NOTE: Pressure can be released by lowering the volume of sterile liquid in the balloon.

6. Balloon is left indwelling at the physician's discretion until hemostasis across the endometrial cavity is achieved.

CAUTION: Device should not be left indwelling for more than twenty-four (24) hours.

NOTE: If the uterus relaxes, the volume in the balloon may be inadequate to prevent a recurrence of bleeding. If so, introduce sterile saline into the balloon until desired volume is achieved.

7. To remove, deflate balloon and gently, but firmly pull on the balloon's stem. A smooth forceps may be used to assist, if necessary, after the cervix is exposed.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.