EN
3

Cook® Balloon Cervical Cannula

Instructions for Use

CS
4

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DA
4

Cook® cervikalkanyle med ballon

Brugsanvisning

DE
5

Cook® Ballon-Cervikalkanüle

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EL
6

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Cook® Cervicale balloncanule

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NO
12

Cook® ballongkanyle for livmorhalsen

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T _ J - C B C C _ R E V O

COOK® BALLOON CERVICAL CANNULA

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Triple lumen polycarbonate cannula with two silicone balloons
- Syringe

INTENDED USE

Used for injection of contrast medium and dyes into the uterus and fallopian tube(s) for hysterosalpingography or evaluation of tubal patency. Lumen allows introduction of catheter or other instruments up to 6.0 French in diameter into the uterine cavity.

CONTRAINDICATIONS

None known

WARNINGS

- Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.
- Do not exceed the recommended balloon volume. Overinflation of this device can cause the balloon to rupture.

PRECAUTIONS

The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

1. Advance the cannula transcervically until the distal balloon is positioned in the lower portion of the uterus.
2. Inflate the distal silicone balloon to a maximum volume of 3 ml with the appropriate medium.

WARNING: Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

WARNING: Do not exceed the recommended balloon volume. Overinflation of this device can cause the balloon to rupture.

3. Inflate the proximal silicone balloon to a maximum volume of 5 ml with the appropriate medium.
4. Contrast medium or dye may now be injected for hysterosalpingography or for confirmation of tubal patency, or other instruments may be introduced via the cannula.
5. Deflate the balloons by syringe aspiration before removing cannula.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.