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EN
3
CS
3
DA
4
DE
4
EL
5
ES
6
FR
6
HU
7
IT
8
NL
8
NO
9
PL
9
PT
10
SV
11
ZH
11

T _ J - G S H C _ R E V O

GOLDSTEIN SONOHYSTEROGRAPHY CATHETER

Used to access the uterine cavity for sonohysterography. The adjustable acorn positioner assists in maintaining the catheter's seal at the external cervical os. The ink band provides a reference mark to promote proper catheter positioning. Large oval sideport facilitates instillation of saline solution. Intended for one-time use.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

This device should not be used in the presence of hemorrhage, active pelvic infection, sexually transmitted disease, profuse bleeding, or pregnancy.

This device should only be used by an experienced clinician.

SUGGESTED INSTRUCTIONS FOR USING GOLDSTEIN SONOHYSTEROGRAPHY CATHETER

1. Pre-fill catheter with saline and remove syringe prior to placement. **NOTE: Syringe is reattached after catheter is positioned.**
2. With cervix exposed, insert catheter into uterus using packing forceps. **NOTE: Reference mark on catheter may be used to aid in positioning.**
3. Advance positioner into cervix using forceps.
4. Instill saline through catheter into uterus.
5. Catheter is removed upon completion of ultrasound procedure.

NOTE: Upon removal of catheter, ensure that positioner is still on body of catheter. If still lodged in patient cervix, remove using forceps.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.