EN
3**Intrauterine Access Balloon Catheter**

Instructions for Use

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Bruksanvisning

T _ K I A B _ R E V 1

INTRAUTERINE ACCESS BALLOON CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

(Components may vary)

- Dual lumen catheter with balloon
- Syringe

INTENDED USE

Used for injection of contrast medium into the uterus and fallopian tube(s) for hysterosalpingography, or injection of methylene blue during laparotomy for tuboplasty and/or tubal anastomosis. The lumen allows introduction of catheters or other instruments up to 6.0 French in diameter into the uterine cavity.

CONTRAINDICATIONS

None known

WARNINGS

- Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.
- Do not exceed recommended balloon volume. Over-inflation of this device can cause the balloon to rupture.

PRECAUTIONS

The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

Note: If applicable, remove the shipping stylet from the balloon catheter prior to use.

1. Advance catheter transcervically until the balloon is positioned in the lower portion of the uterus.

WARNING: Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

2. Inflate the balloon with the appropriate medium.

WARNING: Do not exceed recommended balloon volume. Over-inflation of this device can cause the balloon to rupture.

3. Medium may now be injected for hysterosalpingography or for confirmation of tubal patency, or other instruments may be introduced via the catheter.
4. Deflate balloon by syringe aspiration before removing catheter.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

A. C. Winfield, M.D., Department of Radiology, Vanderbilt University School of Medicine, Nashville, Tennessee.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.