Tenacath™ HSG

HSG katetr Tenacath™

Tenacath™ HSG

Tenacath™ HSG

Tenacath™ HSG

Tenacath™ HSG

Tenacath™ HSG

Tenacath™ HSG

Tenacath™ per isterosalpingografia

Tenacath™ HSG

Tenacath™ HSG

Cewnik Tenacath™ do histerosalpingografii

Tenacath™ para histerossalpingografia

Tenacath™ HSG

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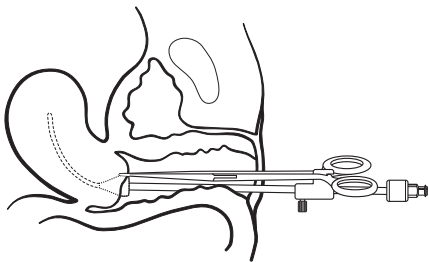
Figure A

TENACATH™ HSG

Used to access the uterine cavity for hysterosalpingography and/or introduction of catheters as large as 9.0 French in diameter. The disposable Tenacath™ is intended for one-time use.

DESCRIPTION

20.0 French radiopaque Tenacath™ 26 cm long with movable bracket and acorn tip.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING TENACATH™ HSG

1. Introduce the Tenacath™ into the cervix until the acorn tip is positioned against the external os of the cervix.
2. While stabilizing the Tenacath™ with one hand, grasp the cervix directly above the os with tenaculum forceps. Insert the forceps locking bar into the Tenacath™ movable bracket. **(Figure A)** Slide the bracket toward the proximal end of the Tenacath™ until the desired tension is achieved. Tighten the thumbscrew to maintain tension.

REFERENCES

J. O. Bergstrom, M.D., Portland, Oregon.

C. M. Barone, M.D., New York, New York.

R. S. Williams, M.D., Department of Obstetrics and Gynecology, The University of Florida College of Medicine, Gainesville, Florida.

R. Gordon, M.D., Department of Radiology, University of California Moffitt Hospital, San Francisco, California.

J. M. LaBerge, D. J. Ponc, R. L. Gordon: "Fallopian Tube Catheterization: Modified Fluoroscopic Technique," Radiology, 176 (1990), 283–284.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.