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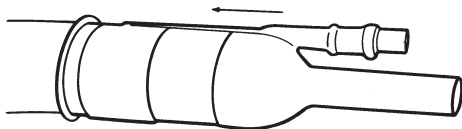
Figure B

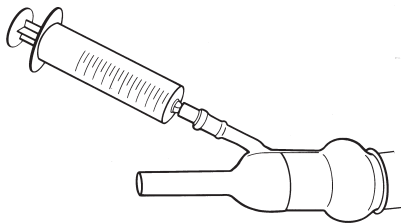
Figure C

NON-ADHESIVE SILICONE CONDOM CATHETER

Used for male external catheterization due to urinary incontinence. The inflatable retention ring secures the catheter and can be easily deflated for removal. Reusable. Intended for single patient use only.

DESCRIPTION

- Non-adhesive silicone condom catheter
- Syringe

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATION

The patient should use this product only after receiving complete instructions from a physician and has a complete understanding of the product and instructions.

SUGGESTED INSTRUCTIONS FOR USING NON-ADHESIVE SILICONE CONDOM CATHETER

NOTE: Patient direct orders can be obtained through Cook with written approval from a physician.

1. Clean the entire penis daily with soap and water. Dry thoroughly. A skin sealant may be applied to the penis if desired.

CAUTION: Do not use this product if skin lesions are present. Seek medical advice before attempting to use.

2. Remove the catheter and syringe from the package.
3. Cuff or roll the sheath down to the inflation valve. **(Figure A)**
4. Place the head of the penis into the catheter. Leave a 1/4 to 1/2 inch space between the catheter tip and the end of the penis to allow proper drainage.
5. Unroll the sheath and position the inflation check valve on top of the penile shaft. The retention ring must be fully exposed and positioned behind the head of the penis. **(Figure B)**
6. Draw 4 to 8 ml of air into the syringe.
7. Firmly press the syringe tip into the inflation check valve. Inflate the retention ring until the catheter feels secure. **(Figure C)** If the catheter does not feel secure or if leakage occurs around the retention ring, add air in increments of 1/2 ml until a good seal is obtained. Firmly grasp inflation check valve and remove syringe.
8. Connect the condom catheter to a drainage bag.
9. Fifteen (15) to twenty (20) minutes after the initial placement of the condom catheter, deflate the retention ring, remove the catheter, and inspect the skin surface. If any redness is noticed, wait until it fades before replacing the catheter. Then replace the catheter and inflate with 1/2 ml less air than previously used. Repeat the procedure until a comfortable yet effective seal is obtained.

CAUTION: Overinflation of the retention ring can impair blood circulation which may not be immediately observable. If any unusual skin irritation, discoloration, localized swelling or pain occurs, contact your physician immediately.

10. Record the volume of air needed to properly inflate the retention ring. Use the same amount of air each time the catheter is placed. When removing the catheter, withdraw the exact amount of air used for inflation. The air left in the retention ring will affect the next inflation.

NOTE: Proper inflation should be checked periodically (every 6 to 8 hours). If gradual balloon deflation is noticed, it may be necessary to flush the inflation check valve with 3 to 4 ml of water to remove debris. Inject water into inflation check valve (with syringe) and then withdraw water. Repeat this process 2 to 3 times. Remove water before reusing. Reinflate with the proper volume of air as previously noted.

CAUTION: The Non-Adhesive Silicone Condom Catheter is limited to single patient use.

CLEANING INSTRUCTIONS

NOTE: It is recommended that two condom catheters be used interchangeably so that a fresh catheter is applied every day.

1. The catheter should be washed with mild soap and lukewarm water, then rinsed well.
2. Hang to dry. A clothes pin may be used to keep the catheter open while drying.
3. Store at room temperature and avoid exposure to direct sunlight.

HELPFUL INFORMATION

1. If the catheter has a tendency to stick together, apply a small amount of baby powder or corn starch to the dry inner surface.
2. Uncircumcised males must leave the foreskin in a natural position over the head of the penis when applying the condom catheter.
3. Catheter life is approximately three (3) weeks with proper care.

If you have any questions concerning the Non-Adhesive Silicone Condom Catheter, contact your physician or Cook®.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.