Rectal Pressure Catheter

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T _ R P C _ R E V O

RECTAL PRESSURE CATHETER

Used for measuring rectal pressure during urodynamic studies. All-silicone construction provides maximum softness and biocompatibility for enhanced patient comfort. Intended for one-time use.

DESCRIPTION

12.0 French all-silicone dual lumen balloon catheter 36 cm long

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING RECTAL PRESSURE CATHETER

CAUTION: Use a water soluble lubricant. Do not use mineral oil.

1. Lubricate balloon, place catheter transrectally and position as desired. Entire length of balloon should be within rectal cavity.

WARNING: Always inflate with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

2. Load a syringe with sterile water and connect to valve marked "FILL."

CAUTION: Do not overinflate the balloon.

3. Instill water into balloon until water flows from the valve marked "T."

IMPORTANT: To obtain accurate pressure measurements, balloon and catheter must be free of air bubbles.

4. Cap the "T" valve until ready to connect catheter to measuring device.
5. Connect "T" valve to measuring device and proceed with studies.
6. Aspirate balloon by syringe prior to withdrawal.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.