Safety Wire Guide Introducer Set

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SAFETY WIRE GUIDE INTRODUCER SET

Used for antegrade placement of a safety wire guide prior to endo-urological procedures. Intended for one-time use.

SET CONTAINS

- 12.0 French radiopaque polyethylene introducer sheath 22 cm long with matched to fit radiopaque polyethylene obturator
- .038 inch (0.97 mm) diameter stainless steel wire guide 100 cm long with 3 cm flexible tip
- 5.0 French radiopaque polyvinylchloride catheter 50 cm long marked in 1 cm increments

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING SAFETY WIRE GUIDE INTRODUCER SET

1. Dilate the nephrostomy tract to 12.0 or 14.0 French over a .038 inch (0.97 mm) diameter wire guide that has been properly placed using the Seldinger technique.
2. Advance the Safety Wire Guide Introducer obturator with sheath over the existing wire guide.
3. Leaving the sheath and wire guide in place, remove the obturator.
4. Insert the .038 inch (0.97 mm) diameter wire guide through the sheath, advancing it about half way down the ureter or into the bladder.
5. Remove the sheath leaving both wire guides.
6. Advance the 5.0 French catheter over the second wire guide; secure the catheter to the skin.
7. Continue dilation of the nephrostomy tract over the original wire guide for the scheduled procedure. The safety wire guide should remain in place until final closure of the tract.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.