Instructions for Use of AQ® Hydrophilic Ureteral Catheters	EN 3
Pokyny k použití hydrofilních ureterálních katetrů AQ®	CS 3
Brugsanvisning til AQ® hydrofile ureterkatetre	DA 4
Gebrauchsanweisung für Ureterkatheter mit AQ® hydrophiler Beschichtung	DE 4
Οδηγίες χρήσης των υδρόφιλων ουρητηρικών καθετήρων AQ®	EL 5
Instrucciones de uso de los catéteres ureterales hidrofílicos AQ®	ES 6
Mode d'emploi des sondes urétérales hydrophiles AQ®	FR 6
Az AQ® hidrofíl ureterális katéterek használati utasítása	HU 7
Istruzioni per l'uso dei cateteri ureterali con rivestimento idrofilo AQ®	IT 7
Gebruiksaanwijzing voor AQ® hydrofiele ureterale katheters	NL 8
Bruksanvisning for hydrofile AQ®-ureterkatetere	NO 9
Instrukcja użycia hydrofilnych cewników moczowodowych AQ®	PL 9
Instruções de utilização de catéteres ureterais hidrófilos AQ®	PT 10
Bruksanvisning för AQ® hydrofila uretärkatetrar	SV 10
AQ®亲水性输尿管导管使 用说明	ZH 11

T _ U C _ R E V O

INSTRUCTIONS FOR USE OF AQ® HYDROPHILIC URETERAL CATHETERS

NOTE: Prior to use, immerse AQ® catheter in sterile water or isotonic saline to allow the hydrophilic surface to absorb water and become lubricious. This will ease placement under standard conditions.

NOTE: These instructions provide a “generic” approach for placing a ureteral catheter. The physician should make appropriate adjustments for ureteral catheters whose configuration does not facilitate over-the-wire placement.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

There are no known contraindications.

1. Pass the appropriate size wire guide into the renal pelvis. **CAUTION: Care must be taken not to pass the wire guide too far into the renal pelvis.**
2. Maintaining wire guide position, pass the ureteral catheter over the wire guide through the ureter and into the renal pelvis.
3. Remove wire guide and then cystoscope while maintaining the catheter position.
4. Remove catheter and dispose.

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Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.