EN
4

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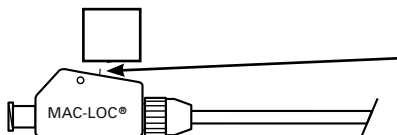
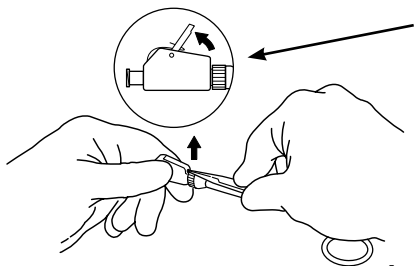
ULTRATHANE® SUPRAPUBIC SET WITH MAC-LOC®

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Ultrathane® suprapubic catheter with Mac-Loc® locking loop mechanism
- Cannula assembly with round-tip obturator
- Needle obturator with trocar tip
- Clear polyvinylchloride connecting tube with one-way stopcock and drainage bag connector
- Disposable scalpel
- StatLock® catheter securing device

INTENDED USE

The Ultrathane® Suprapubic Set is used to provide bladder drainage by percutaneous placement of a loop catheter.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Periodic evaluation is advised; the catheter must not remain indwelling more than four weeks.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.
- To avoid damage to the monofilament, care is necessary when placing the cannula assembly into the catheter.

INSTRUCTIONS FOR USE

1. Anesthetize the skin overlying the puncture site.
2. While gently straightening the retention loop, carefully advance the cannula assembly and round-tip obturator into the catheter until the loop is fully straightened. Secure the cannula assembly with the Luer lock.
3. Pull the monofilament taut.
4. Remove the round-tip obturator and discard.
5. Insert the needle obturator and secure with the Luer lock.
6. Fill the bladder with sterile saline, if not already distended by urine.
7. Make a 2 mm incision in the skin approximately 2.5 to 5.0 cm above the symphysis pubis (below the palpable bladder dome).
8. Insert the catheter *vertically* into the bladder on patients without previous pelvic surgery, or at a 30° angle toward the symphysis pubis on patients with previous pelvic surgery.
9. Remove the needle obturator and observe for urine flow. If bladder pressure is low and urine fails to flow spontaneously through the needle, aspirate urine to ascertain that the catheter tip is in the bladder. Advance the catheter no more than 4 to 5 cm beyond the first sign of bladder fluid.
10. Once the catheter position has been confirmed, carefully remove the cannula assembly while gently advancing the catheter several more centimeters. The catheter must be well within the bladder to ensure proper formation of the retention loop.
11. Lock the retention loop in place. **NOTE:** See Mac-Loc® locking loop mechanism instructions at the end of these instructions.
12. Attach the supplied connecting tube to the catheter and a drainage bag, and allow the bladder to empty.

- Retract the catheter shaft until the loop is felt against the bladder dome. Advance the catheter approximately 2 cm back into the bladder to allow for normal movement.
- Secure the catheter shaft and connecting tube using the enclosed StatLock® catheter securing device.

Locking the Mac-Loc Locking Loop Mechanism

- Stabilize the Mac-Loc catheter hub assembly with one hand and pull back on the monofilament to form the distal catheter loop configuration (**Fig. 1**).
- While maintaining traction on the monofilament, push the locking cam lever down until a distinct “snap” is felt. The catheter’s distal loop is now locked into position (**Fig. 2**).
- Trim off the excess monofilament (**Fig. 3**).

Unlocking the Mac-Loc Locking Loop Mechanism

- While stabilizing the Mac-Loc catheter hub assembly with one hand, position the end of a small, sterile straight or curved forceps into the Mac-Loc release notch.
- Pry upward until the locking cam lever is free (**Fig. 4**). It is now possible to remove the catheter.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

StatLock® is a registered trademark of Venetec International, Inc.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.