EN
3

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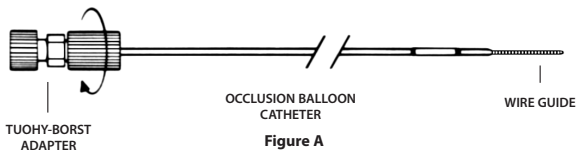
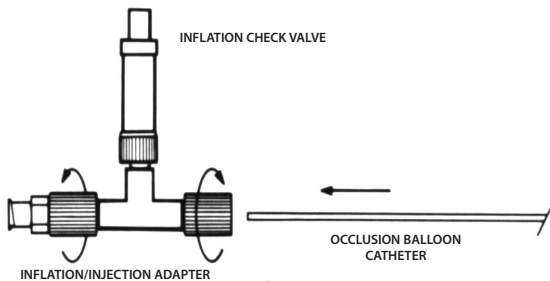
Figure B

UPJ OCCLUSION BALLOON CATHETER SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Set Contains (*set components may vary*)
- Radiopaque balloon catheter
- Appropriate diameter PTFE coated stainless steel wire guide 80 cm long
- Removable inflation/injection adapter
- Syringe

INTENDED USE

Used to temporarily occlude the ureteropelvic junction to prevent stone fragments from entering the ureter during percutaneous lithotripsy, and for injection of contrast medium. The removable inflation/injection adapter allows the cystoscope to be removed after the catheter is in place.

CONTRAINDICATIONS

There are no known contraindications.

WARNINGS

- Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

PRECAUTIONS

- Using excessive pressure to inflate the balloon on this device can cause the balloon to rupture. Inflation of the balloon 2 or 3 times with an appropriate inflation medium prior to use will allow easier inflation of the balloon.
- Only water soluble lubricants are recommended for use with the occlusion balloon catheter. Use of other than water soluble lubricants may cause the inflated balloon to burst.
- Do not overinflate the balloon.
- Refer to product label or the inflation check valve on the balloon device for appropriate balloon volume.
- To prevent the balloon from rupturing, avoid excessive pressure. The amount of inflation may be determined fluoroscopically.
- This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease

INSTRUCTIONS FOR USING UPJ OCCLUSION BALLOON CATHETER SET

1. Remove the TFE sheath which protects the balloon section of the UPJ Occlusion Balloon Catheter; discard the sheath.
2. Secure the wire guide to the catheter by tightening the Tuohy-Borst adapter. **(Figure A)** The wire should protrude approximately 1 cm to 2 cm from the occlusion balloon catheter tip to act as a filiform during placement.
3. Under x-ray or fluoroscopic control, cystoscopically pass the UPJ Occlusion Balloon Catheter with wire guide into the ureter and up to the UPJ.
4. After positioning and before inflating the balloon, remove the wire guide and then remove the cystoscope.
5. Slide the inflation/injection adapter onto the distal end of the occlusion balloon catheter until the tip meets the Luer lock adapter. Tighten the caps on both ends of the adapter. **(Figure B)**

6. Fill the syringe with dilute (25% solution) contrast medium. Eliminate air from the syringe.
7. Press the syringe tip into the inflation check valve. Using light to moderate thumb pressure, inject the appropriate amount of contrast medium.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.