EN 3	uVue™ HSG/SHG Catheter Instructions for Use
CS 4	Katetr uVue™ HSG/SHG Návod k použití
DA 6	uVue™ HSG/SHG-kateter Brugsanvisning
DE 7	uVue™ HSG/SHG-Katheter Gebrauchsanweisung
EL 9	Καθετήρας uVue™ HSG/SHG Οδηγίες χρήσης
ES 10	Catéter uVue™ HSG/SHG Instrucciones de uso
FR 12	Cathéter uVue™ pour HSG/SHG Mode d'emploi
HU 14	uVue™ HSG/SHG katéter Használati utasítás
IT 15	Catetere per HSG/SHG uVue™ Istruzioni per l'uso
NL 17	uVue™ HSG/SHG-katheter Gebruiksaanwijzing
NO 18	uVue™ HSG/SHG kateter Bruksanvisning
PL 20	Cewnik uVue™ HSG/SHG Instrukcja użycia
PT 21	Cateter uVue™ HSG/SHG Instruções de utilização
SV 23	uVue™ HSG/SHG-kateter Bruksanvisning

uVue™ HSG/SHG CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Soft polymer catheter with internal stylet and silicone balloon
- Syringe

INTENDED USE

The uVue HSG/SHG Catheter is intended to access the uterine cavity for sonohysterography (SHG), hysterosalpingography (HSG) and other diagnostic studies.

CONTRAINDICATIONS

This device should not be used in the presence of hemorrhage, active pelvic infection, sexually transmitted disease, profuse bleeding, pregnancy, recent cesarean section, or if the patient currently has or suspects she may have an intrauterine device.

WARNINGS

- Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.
- Do not overinflate. Using excessive volume to inflate the balloon on this device can cause the balloon to rupture. Refer to product label or the inflation line on the balloon device for appropriate balloon volume.

PRECAUTIONS

- This procedure should only be performed by a physician or properly licensed practitioner trained in diagnostic and interventional techniques. Standard techniques for SHG and HSG procedures should be employed.
- Use of the uVue HSG/SHG Catheter in postpartum patients or patients with a recent uterine perforation should be determined at the physician's discretion. Risk of uterine wall injury may be increased in these patients.

INSTRUCTIONS FOR USE

Sonohysterography (SHG)

1. Form the catheter to desired shape, if applicable.
2. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
3. Cleanse the cervix with an appropriate cleansing solution.
4. Prime the catheter with saline.
5. Insert the catheter tip through cervix and into the uterus.
6. Fill the syringe with sterile saline solution. Eliminate air from the syringe.
7. Inflate the balloon by instilling sterile saline solution and close the stopcock once desired volume is injected.
8. Pull back gently on the catheter shaft to seat the inflated balloon against the internal cervical os.
9. Inject sufficient medium through the catheter into the uterus.
10. Perform ultrasound procedure.
11. Deflate the balloon.
12. Take a ultrasound final examination of the lower uterus.
13. Remove the catheter from the cervix when examination is complete.

Hysterosalpingography (HSG)

1. Form the catheter to desired shape, if applicable.
2. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
3. Cleanse the cervix with an appropriate cleansing solution.
4. Prime the catheter with medium.
5. Insert the catheter tip through the cervix and into the uterus.
6. Fill the syringe with sterile saline solution. Eliminate air from the syringe.
7. Inflate the balloon by instilling sterile saline solution and close the stopcock once desired volume is injected.
8. Pull back gently on the catheter shaft to seat the inflated balloon against the internal cervical os.
9. Inject sufficient contrast medium through the catheter into the uterus.
10. Perform fluoroscopic procedure.
11. Deflate the balloon.
12. Take a fluoroscopic final examination of the lower uterus.
13. Remove the catheter from the cervix when examination is complete.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2014
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.